Prior to use, carefully read the instructions for use.

**EN**

## CERAMIC PRIMER II

PRIMER FOR CERAMIC AND COMPOSITE BONDING

CERAMIC PRIMER II is a priming agent designed to treat the adherend surface of ceramic and composite prostheses to promote a strong bond to resin-based materials as resin-based cements in case of luting procedures or direct composites in case of repairs.

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

- Priming of the adherend surfaces of indirect restorations made of ceramics (glass ceramics, zirconia and alumina), hybrid ceramics (e.g. Cerasmart) and composite before luting procedures.
- Intra-oral and extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics and composites.

#### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

#### DIRECTIONS FOR USE

- For luting procedure
  - Treat the bonding surfaces of prosthesis according to the respective manufacturer’s instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath or with alcohol, then rinse and dry with an oil-free air syringe.
  - Bottle type:** Dispense one drop of CERAMIC PRIMER II into a dispensing dish.
**Unit dose type:** Unscrew cap of unit dose counterclockwise. Make sure to identify the top and bottom of the unit dose capsule before opening.
  - Apply a thin layer to the bonding surface of the ceramic or composite prosthesis using a micro-tip applicator and dry with an air syringe.
  - Apply suitable resin-based luting cement to the primed surface of the prosthesis. For these steps, refer to the manufacturer’s instructions for use of the corresponding luting cement.

#### II. For Intra-oral Repair

- Roughen the bonding surface of ceramic or composite using a coarse diamond bur or a carborundum point.
- Treat the bonding surfaces of the prosthesis according to the respective manufacturer’s instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
- Bottle type:** Dispense one drop of CERAMIC PRIMER II into a dispensing dish.
**Unit dose type:** Unscrew cap of unit dose counterclockwise. Make sure to identify the top and bottom of the unit dose capsule before opening.
- Apply a thin layer to the fractured surface of the ceramic or composite prosthesis using a micro-tip applicator and dry with an oil-free air syringe.
- Continue the repair by using a light-cured adhesive (e.g. G-aenial Bond) and a light-cured composite (e.g. G-aenial), referring to their respective manufacturer’s instructions for use.

Nota:

If the treated adherend surface is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the manufacturer’s instructions and re-apply CERAMIC PRIMER II according to the instructions above.

#### STORAGE

Recommended for optimal performance, store at room temperature (4-25°C, 39.2-77.0°F). Keep away from direct sunlight. Shelf life: 2 years from date of manufacture.

#### PACKAGE

- CERAMIC PRIMER II, 1 x bottle of 3mL
- CERAMIC PRIMER II, 10 x unit dose of 0.1mL

Nota:

All packaging types may not be available in all countries.

#### CAUTION

- CERAMIC PRIMER II is flammable and volatile. Do not use near naked flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
- In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Rinse immediately after luting or repairing.
- In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Replace the cap of bottle immediately after use. If dew condensation is present on the bottle of CERAMIC PRIMER II, immediately wipe it off.
- Unit dose is disposable. **Do not recap or reuse to avoid cross contamination between patients.**
- If the cap of the unit dose is too tight to open, do not try to open it with force to avoid accidental contact of the liquid with skin and eyes. Use another unit dose.
- Take care to avoid ingestion of the material.
- Do not mix with any other primers.

Last revised: 02/2015

MANUFACTURED by
**GC DENTAL PRODUCTS CORP.**
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
**GC AMERICA INC.**
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED by
**GC CORPORATION**
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU : GC EUROPE N.V.**
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium
TEL: +32 16 74 10 00

**GC AMERICA INC.**
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

**GC ASIA DENTAL PTE. LTD.**
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

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