# COE-SOFT™

---

RESILIENT DENTURE LINER

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CE 0086

**GC**®

MANUFACTURED by  
GC AMERICA INC.  
3737 West 127th Street  
Alsip, Illinois 60803 U.S.A.

DISTRIBUTED by  
GC EUROPE N.V.  
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium

**GC**®

GC AMERICA INC.  
ALSIP, IL 60803 U.S.A.

# COE-SOFT™

GB

## RESILIENT DENTURE LINER

Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

### CONTRAINDICATIONS:

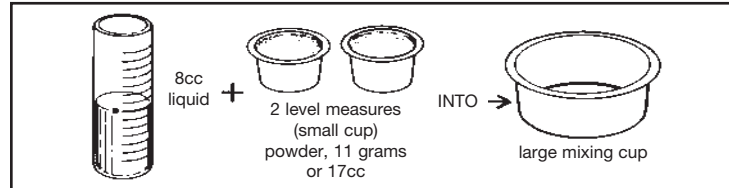
Patients who have shown sensitivity to the material. In case of allergy refer to a physician.

### RECOMMENDED INDICATIONS:

A temporary lining for acrylic dentures. For use in chairside procedures.

### DIRECTIONS FOR USE:

- Preparation of the denture:** Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT. Note: COE-SOFT will not adhere to surfaces coated with COE LUBRICANT.
- Preparation of COE-SOFT:** Recommended powder / liquid ratio is 11g powder to 8ml liquid. Pour the liquid into the large mixing cup. Then add the powder slowly. Stir mixture thoroughly for 30 seconds. A suitable spatula is provided for this purpose. To avoid introducing bubbles into the mixture, do not spatulate for more than 30 seconds. Do not whip.

- Application:** Spread the mixture of COE-SOFT over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion, and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.
- Finishing:** When curing is complete, trim away excess. For smoothing the edges use a hot spatula.
- Patient Advice:** Advise the patient NOT to use a brush or abrasive (such as toothpaste) on the lining. Cleaning is best achieved by holding it under cold running water and wiping with wet cotton.

### STORAGE:

Store in a dry location at room temperature (70° to 77°F; 21° to 25°C). (3 year shelf-life guarantee).

### PACKAGES:

344001 COE-SOFT Professional Package	344003 COE-SOFT Economy Package
344002 COE-SOFT Powder, 6oz (170g)	344004 COE-SOFT Powder, 5lb (2.27kg)
344091 COE-SOFT Liquid, 6oz (177ml)	344092 COE-SOFT Liquid, 32oz (946ml)

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