Prior to use, carefully read the instructions for use.

EN

COCOA BUTTER

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24hours after placement.
- To prevent cement from sticking to adjacent teeth and instruments.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

HOW TO USE

- Place glass ionomer cement according to the manufacturer's instructions.
- Extrude the required amount of material from the tube. Using a spatula or cotton pellet, apply uniformly to the area to be moisture proofed / isolated.
- After use, immediately replace cap.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life: 5 years from date of manufacture)

PACKAGE

Tube of 10g
Tube of 2g

Note:
All packaging types may not be available in all countries.

CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 06/2015

GC

MANUFACTURED by
GC DENTAL (SUZHOU) CO., LTD.
Qing Qiu Street, Suzhou Industrial Park, Suzhou, 215126 P. R. China

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Hasarode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05 Singapore 528729
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9316 4499

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