



Fuji CAPSULE goes gold


GC Gold Label CAPSULE

Celebrating **over one billion** glass ionomer restorations

GC

Now branded GC Gold Label CAPSULE, our bioactive glass ionomer capsules are the best choice for you and your patients

Here's why GC's market-leading glass ionomer cements are celebrating over 1 billion restorations worldwide:

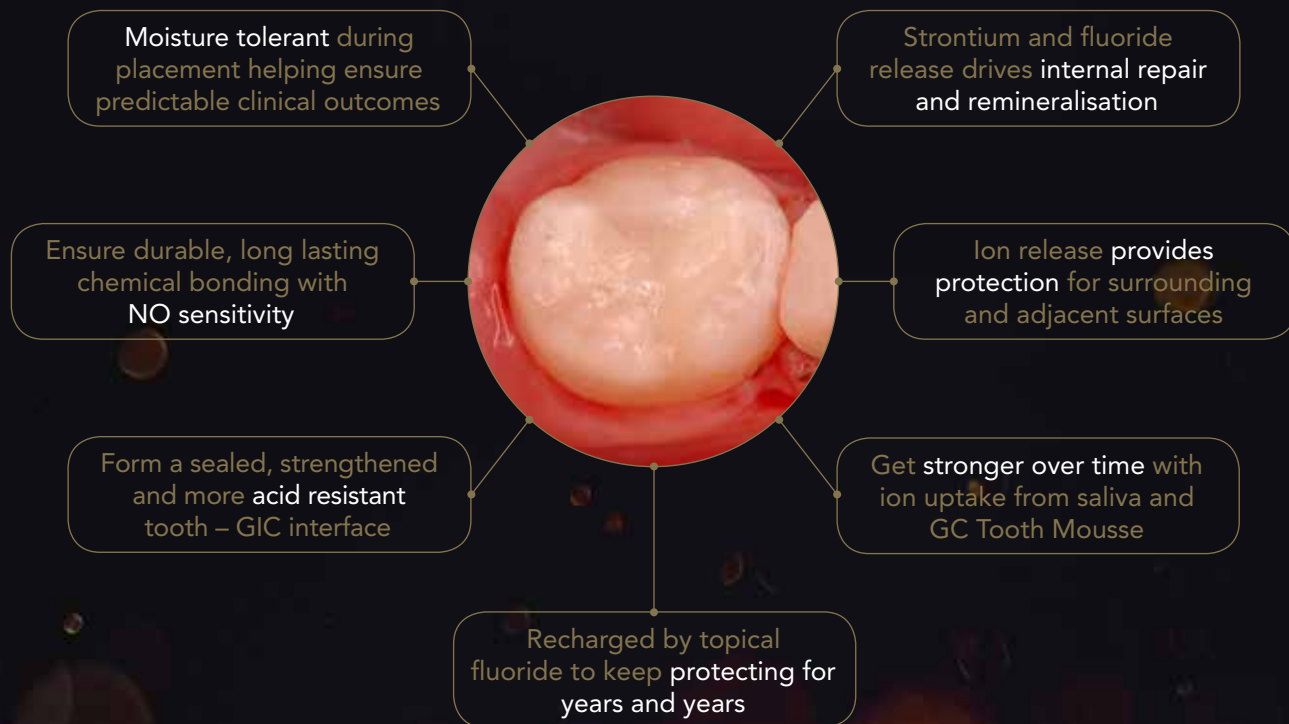

Step up from hand-mixing to the highest performance level with GC Gold Label CAPSULE



Fast and simple

Direct injection speeds placement with fewer compromises in challenging clinical cases.



Ideal consistency

The perfect mixing ratio optimises flow, cavity adaptation, reduces voids and ensures the strongest chemical adhesion.



Convenient and consistent

The same predictable result each and every time.



Adjustable working time

Capsules can be refrigerated to extend working time.



Higher strength

Exact proportioning of powder & liquid assures the highest physical properties for stronger, more durable restorations.



Simple activation



A: Shake or tap to loosen powder.



B: Depress plunger, hold down 2 seconds.



C: Mix for 10 seconds.



D: Two clicks to prime capsule then syringe slowly.



GC Gold Label IX EXTRA CAPSULE

The benchmark **high strength**, radiopaque glass ionomer restorative for **posterior** use.

RESTORATION



Dr J Lucas



Dr J Lucas

Features:

- Extra protection with high fluoride release.
- Extra translucency for improved aesthetics
- Bulk placement with up to 60 seconds working time*
- Extra fast setting so final finishing begins just 2½ minutes after start of mix
- Clinically proven performance

Indications:

- Posterior restorations in primary and permanent (non-load bearing) dentition
- As a base or core buildup under direct and indirect restorations
- Cervical restorations



*Temperature dependent

GC Gold Label II LC CAPSULE

The benchmark **light-cured** glass ionomer cement for **aesthetic** restorations.

RESTORATION



Dr. J. Lucas

Features:

- Outstanding aesthetics
- Exceptional adhesive performance in class 5 restorations
- Up to 2 minutes manipulation time*
- Immediate finishing after 20 seconds light-curing
- Clinically proven performance
- Confidence of tri-cure, resin-reinforced glass ionomer technology

Indications:

- Cervical restorations
- Proximal restorations
- As a base or core buildup under direct and indirect restorations



*Temperature dependent

GC Gold Label VII CAPSULE

The **benchmark** for glass ionomer **protection**.

PROTECTION



Dr J Lucas

Indications and Features:

- Protects any tooth surface at risk of acid erosion or decay, e.g. erupting molars, root surfaces.
- Ideal for caries stabilisation, deep lesions and endo-sealing
- Up to 90 seconds working time* with a 2½ minute snap set
- Flowable consistency
- Highest fluoride release



GC Gold Label VII EP CAPSULE

Glass ionomer protection, **enhanced with RECALDENT™ (CPP-ACP)**.



Dr G Knight

Indications and Features:

- Highest level of protection with bioavailable calcium, phosphate and fluoride release
- Reduces sensitivity, e.g. molar hypomineralisation
- Enhanced remineralisation capabilities
- Superior biofilm inhibition



*Temperature dependent

GC Gold Label 1 CAPSULE

The **benchmark** glass ionomer **luting cement**.

CEMENTATION



Indications and Features:

- Routine cementation for metal and zirconia crown & bridge
- Smooth consistency for easy seating
- Remove excess approx 90 seconds after seating*, final set after 4½ minutes
- Proven, long lasting performance



GC Gold Label PLUS CAPSULE

Resin-reinforced, for **stronger** glass ionomer **cementation**.



Indications and Features:

- Proven performance for metal, zirconia & lithium disilicate
- Creamy consistency, 10 micron film thickness
- Remove excess approx 60 seconds after seating*, final set after 4¼ minutes
- No sensitivity



*Temperature dependent



GC Gold Label IX EXTRA CAPSULE
Box 30 capsules (0.14mL each)
Shades A2, A3, A3.5



GC Gold Label VII CAPSULE
Box 30 capsules (0.13mL each)
Shades Pink, White



GC Gold Label I CAPSULE
Box 30 capsules (0.19mL each)



GC Gold Label II LC CAPSULE
Box 30 capsules (0.10mL each)
Shades A1, A2, A3, A3.5, A4



GC Gold Label VII EP CAPSULE
Box 30 capsules (0.13mL each)
Shades Pink, White



GC Gold Label PLUS CAPSULE
Box 30 capsules (0.19mL each)



HSM-1 Mixer



GC CAPSULE APPLIER III



DENTIN CONDITIONER
25g (23.8mL)



G-COAT PLUS
4mL



Since 1921
Towards Century of Health

GC Asia Dental Pte Ltd
T: +65 6546 7588
F: +65 6546 7577

W: sea.gcasiadental.com
E: gcasia@gc.dental