Prior to use, carefully read the instructions

**EN**

## GC Initial™ INmetalbond

Buffer between GC Initial metal ceramics and dental alloys.

**GC Initial INmetalbond**

INmetalbond is used as a thin layer between the alloy and the first Opaque layer. The INmetalbond blocks the escaping metal oxides and neutralizes differences in the expansion coefficient. The bonder does not increase the bonding strength as such but allows a wider span of CTE compatibility.

**Recommended indications**

Can be used on all PFM alloys, precious as well as non-precious.

**Instruction For Use**

- Metal framework needs to be prepared according to manufacturer’s instructions.
- Oxidation firing again according to manufacturer’s instructions.
- Stir paste before using.
- Apply the Bonder in a thin layer but mask the metal completely.

**IMPORTANT:**

- To change the consistency of the INmetalbond paste, use the “GC Initial Paste Opaque Thinner”.
- Only use it in very small quantities.
- To avoid drying out, close the cap after using.
- The fired Bonder should have a yellowish, slightly shiny surface.

**NOTE:**

- The colour of the fired bonder can vary depending on the composition of the alloys.
- When using NPA (non-precious alloys) with the INmetalbond, the final temperature of the Opaque Washbake does not need to be increased by 20°C anymore. e.g.: 1st Opaque Firing on NPA: GC Initial MC = 940°C GC Initial LF = 830°C

**Firing instructions**

Preheating Temp.	Drying Time	T° increase	Vacuum	Final Temp.	Holding Time
550°C	6min	80°C/min	Yes	980°C	1 min no vacuum

**Storage**

Recommended for optimal performance, store in a cool, dry place.

**Shelf life**

GC Initial INmetalbond: 5 years from date of manufacture

**Packages**

GC Initial INmetalbond, 2x4gr.

**CAUTION**

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: http://www.gceurope.com or for The Americas http://www.gcamerica.com They can also be obtained from your supplier.

Last revised: 11/2017

**CE** 0086

**'GC.'**

MANUFACTURED by

**GC EUROPE N.V.**  
Research-Park Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

DISTRIBUTED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**GC EUROPE N.V.**  
Research-Park Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

**GC AMERICA INC.**  
3737 West 127th Street, Alsip, IL 60803 U.S.A. - TEL: +1-708-597-0900, www.gcamerica.com

**GC SOUTH AMERICA**  
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL  
CEP: 02022-051 - TEL: +55-11-2925-0965 - CNPJ: 08.279.999/0001-61  
RESP. TÈC: Mayara de Santis Ribeiro - CRO/SP 105.982

**GC ASIA DENTAL PTE. LTD.**  
11 Tampines Concourse, #03-05, Singapore 528729 - TEL: +65 6546 7588

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