Prior to use, carefully read the instructions for use.

EN

## GC PLIERS

### PLIERS FOR REMOVAL OF PROSTHETIC APPLIANCES

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

1. Removal of luted temporary prosthetic appliances.
2. For removal of all types of prosthetic appliances during trial fitting.

#### DIRECTIONS FOR USE

1. REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES
  - a) Use the GC PLIERS without the rubber pads.
  - b) Hold the cervical area of the crown with the tips of the pliers.
  - c) Twist the crown alternatively towards the mesial and distal direction.
  - d) Remove the crown in a labiolingual direction.
2. REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING
  - a) Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers.
  - b) Place emery powder on the surface of the pads in order to improve the grip.
  - c) Continue with 1 b).

#### CLEANING

Cleaning of the GC Pliers made out of stainless steel, can be accomplished by hand scrubbing, ultrasonic cleaning or thermodesinfection with a non-corrosive, neutral detergent. For further details consult the respective manufacturer's instructions.

#### STERILISATION

GC Pliers may be routinely sterilized by the following methods : autoclave, chemiclave or dry heat.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.