Prior to use, carefully read the instructions for use.

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DIRECTIONS FOR USE

- PREPARATION**
 - Check the patient’s oral conditions and the denture in use. Adjust misfitting areas with a carbide bur, clean and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain the elasticity. Apply cocoa butter or petroleum jelly to the areas where the denture should not be relined.

- POWDER /LIQUID DISPENSING**

The standard powder/liquid ratio is 2.4 g/2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup.

Note:

- Using less liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.
- Powder may form aggregates in the bottle under certain storage conditions. This will not affect the function.
- Close the bottles of powder, liquid and coating agent tightly immediately after use.

- MIXING**

Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.

- APPLICATION**

1) The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only on the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand.

2) Apply the mixture evenly to the fitting surfaces of the prepared denture.

Note:

- To avoid sticking of the material, lubricate fingertips or instruments with water. Do not use lotion or petroleum jelly.
- Prevent excess from flowing into the throat.
- Seat the denture in the mouth. Ask the patient to gently occlude in the centric occlusion and move the lips and mandibles to muscle trim. After keeping the denture for about 5 minutes in this position, remove and inspect for excess and deficiencies.
- Remove any excess using a sharp instrument.

- FINISHING**

- Wash away saliva from the relined denture under running water. Dry with an air syringe.
- Apply the coating agent to the relined surfaces of the denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 4-5 minutes.

Note:

To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.

- Reset the denture in the mouth.

Note:

- Immersion of the denture in water before seating in the mouth will reduce unwanted odor.
- Instruct the patient to clean the relined areas with fingers under running water. The use of a brush may damage the relined areas. Care should be taken not to use denture cleaning agents for a prolonged period of time.

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STORAGE

Store in a cool, dark place (4-25°C) (39.2-77.0°F) away from direct sunlight and high temperature.

Shelf life:

Powder, Coating agent - 2 years from date of manufacture

Liquid - 3 years from date of manufacture

COLORS

Live pink, White

PACKAGES

- 1-1 Package

Kit contains: Powder 90g (1), Liquid 90g (101mL) (1), Coating agent 12g (15mL) (1), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1), Rubber cup (Large) (1)

- Refill Packages

- GC TISSUE CONDITIONER powder 90g (in each shade)
- GC TISSUE CONDITIONER liquid 90g (101mL)
- GC TISSUE CONDITIONER coating agent 12g (15mL)

CAUTION

- The liquid is flammable. Avoid sources of ignition. Do not keep or use near fire or high temperature.
- Use in a well ventilated place. Replace cap immediately.
- Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
- Avoid direct contact of liquid, mixture or coating agent on skin. In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.
- Coating agent may turn yellow with time, but this will not affect its function.
- Do not mix with other products.

Last revised : 09/2008

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlan 33,
B-3001 Leuven, Belgium TEL: +32. 16. 39. 80. 50.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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Prior to use, carefully read the instructions for use. **EN**

GC TISSUE CONDITIONER

TISSUE CONDITIONING AND RELINING MATERIAL

For use by or under the supervision of a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Tissue conditioning.
- Temporary relining.
- Functional impression taking.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- PREPARATION**

Check the patient’s oral conditions and the denture in use. Adjust misfitting areas with a carbide bur, clean and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain the elasticity. Apply cocoa butter or petroleum jelly to the areas where the denture should not be relined.
- UNDER LQUID DISPENSING**

In a rare case the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

2.1. **WORKING RATIO**
Mix with distilled water in a 2:4 g/2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup.

Note:

 - Using less liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.
 - Powder may form aggregates in the bottle under certain storage conditions. This will not affect the function where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand.
 - Close the bottles of powder, liquid and coating agent tightly immediately after use.
- MIXING**

Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.
- APPLICATION**
 - The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent to the brush only in the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand.
 - Apply the mixture evenly to the fitting surfaces of the prepared denture.
 - Note:
 - After avoid sticking of the material, lubricate fingertips or instruments with water. Do not use lotion or petroleum jelly.
 - Do not use the material in contact with the skin.
 - Seat the denture in the mouth. Ask the patient to gently occlude in the centric occlusion and move the lips and mandibles to muscle trim. After keeping the denture for about 5 minutes in this position, repeat and inspect for excess and deficiencies.
 - Remove any excess using a sharp instrument.
 - FINISHING**
 - Wash away saliva from the relined denture under running water. Dry with a brush.
 - Apply the coating agent to the relined surfaces of the denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 4-5 minutes.
 - Repeat the denture in the mouth.

To improve the margin seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.

- Repeat the denture in the mouth.

Note:

- Immersion of the denture in water before seating in the mouth will reduce unwanted odor.
- Immerse the patient to clean the relined areas with fingers under running water. The use of a brush may damage the relined areas. Do not use tooth powder to use denture cleaning agents for a prolonged period of time.

- REMOVAL**

It is recommended to remove the material or replace it with a new one after a few days or weeks depending on each clinical case. GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.

STORAGE
Store in a cool, dark place (4-25°C) (39.2-77.0°F) away from direct sunlight and high temperature.

COLORS
Live pink, White

PACKAGES
1. 1-1 Package
KIT content: Powder 90g (1), Liquid 90g (101mL) (1), Coating agent 12g (15mL) (1), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1), Rubber cup (Large) (1)
2. Retail Package
a. GC TISSUE CONDITIONER powder 90g (in each shade)
b. GC TISSUE CONDITIONER liquid 90g (101mL)
c. GC TISSUE CONDITIONER coating agent 12g (15mL)

CAUTION
1. The liquid is flammable. Avoid sources of ignition. Do not keep or use near fire or high temperature.
2. Use in a well ventilated place. Replace cap immediately.
3. Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
4. Avoid direct contact of liquid, mixture or coating agent on skin.
In case of contact, wipe with a cotton pellet, but this will not affect its function.
5. Coating agent may turn yellow with time, or it should not affect its function.
6. Do not mix with other products.

SHelf life
Powder, Coating agent - 2 years from date of manufacture
Liquid - 3 years from date of manufacture

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MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0904, Japan

DISTRIBUTED BY
GC CORPORATION
7-1 Heiwaumachino, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
De Weertsestraat 22, 3000 Leuven, Belgium
TEL: +32 16 39 30 50

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