Miracle Mix® CAPSULE

Prior to use, carefully read the instructions for use.

RADIOPAQUE SILVER ALLOY-GLASS IONOMER CEMENT RESTORATIVE IN CAPSULES

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Core build-up.
2. Restoration of primary teeth.
3. Cases where radiopacity is required.
4. As a base or liner.
5. Class I, limited Class II, temporary fillings, etc.


CONTRAINDICATIONS

1. Pulp capping.
2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE





1. CAVITY PREPARATION

- a) Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
- b) Wash and dry but **DO NOT DESICCATE**. GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) is recommended to remove the smear layer (Fig. 1).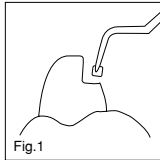



2. CAPSULE ACTIVATION / MIXING

- a) Before activation shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).
- b) To activate the capsule push the plunger until it is flush with main body (Fig. 3).
- c) **Immediately** place the capsule into a metal GC Capsule Applier and click the lever once (Fig. 4). The capsule is now activated.
Note :
The capsule should be activated just before mixing and used **immediately**.
- d) **Immediately** remove the capsule and set it into a mixer (an amalgamator) and mix for 10 seconds at high speed (+/-4,000 RPM) (Fig. 5).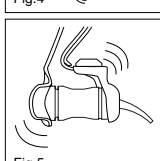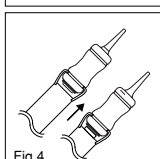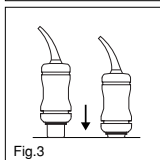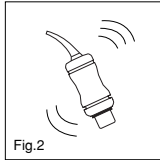


3. RESTORATIVE TECHNIQUE

- a) **Immediately** remove the mixed capsule from the mixer and load it into the GC Capsule Applier. Time of initial extrusion after mixing : 10 seconds maximum.
- b) Make two clicks to prime the capsule then syringe (Fig. 6).
The working time is 1 minute from start of mixing at 23°C (73.4°F). Higher temperature will shorten working time.
- c) Remove surface moisture but **DO NOT DESICCATE**.
- d) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 7). Avoid air bubbles.
Note :
 - 1) To adjust the direction of the nozzle, hold the applicator with the capsule towards you and turn the capsule body.
 - 2) To remove the used capsule, push the applicator release button. Twist the capsule and pull upwards.
- e) Form the contour.
- f) Setting time is 5'00" from start of mixing.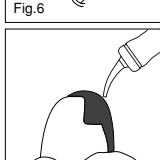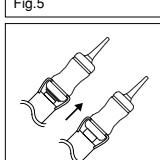

4. FINISHING

Use standard techniques (Fig. 8).

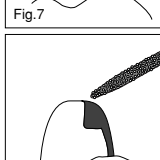STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life : 2 years from date of manufacture).

PACKAGE

Package : 50 capsules.
Average contents per capsule : 0.52g silver alloy ionomer cement and 0.11g (0.09mL) liquid.
Minimum net volume of mixed cement per capsule : 0.09mL
Option : GC Capsule Applier (1 piece).

CAUTION

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.